


CH3A-2205-IN Alarm Clock Charger

Part of the SA3000 Paging System for deaf and deaf-blind people


Charging Plug

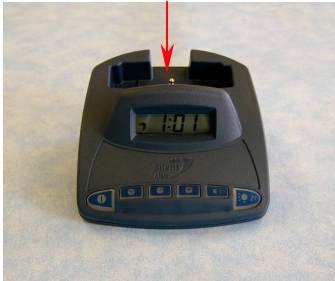

Minute Advance
Hour Advance
Alarm On/Off



Pager docked in charging slot

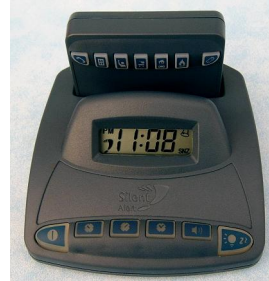

Power Supply



Power Supply
Plug

Pillow Pad
Plug

Pillow Pad



Getting Started

1. Locate the Alarm Clock Charger on a bedside table and connect the power supply and pillow pad.
2. Plug the Power Supply into a 13amp mains socket. Numbers on the alarm clock display will appear indicating that the unit is on. Leave the charger switched on at all times as it includes an emergency back-up battery.
3. Ensure the pager unit is switched on and dock it onto the charging plug ensuring the pager is pushed fully down. If there is sufficient charge in the battery the pillow pad will activate three times to confirm that the pager has been docked correctly. In any event charge the pager for 24 hours.
4. Install the universal monitor units (Part No. UM3A-2212-EU) according to their instructions.
5. Whilst the Pager is in the Charger slot operate each universal monitor in turn. The Pager will respond by lighting the relevant key and activating the pillow pad/strobe light.
6. To reset the pager and stop the vibration/flashing, press and hold the key that is lit. Each signal will automatically cancel after 1-2 minutes except the fire key, which must be reset by the user.
7. The battery key on the pager will not flash whilst it is in the charger.
8. The Pager must be recharged each night and if not in use for more than 24 hours should be either placed on charge or switched off.
9. Set the clock time using the clock set, hour and minute advance keys. Use the alarm set, hour and minute advance keys to set the alarm time. Switch the alarm on or off using the alarm On/Off key. The pillow pad will activate three times when the alarm is switched on and once when it is switched off. When the alarm sounds the pillow pad will activate whilst the display light, Bell and SNZ symbols flash. To cancel the alarm, press the alarm On/Off key. To delay the alarm for 4 minutes press the snooze/light key.

DO NOT IMMERSE THE ALARM CLOCK CHARGER IN WATER OR ANY OTHER LIQUID. DO NOT SUBJECT THE UNIT TO EXCESSIVE MECHANICAL SHOCK. THE ALARM CLOCK CHARGER SHOULD BE PLACED ON A SMOOTH HARD SURFACE AND AWAY FROM OTHER OBJECTS TO ENSURE CORRECT VENTILATION. DO NOT COVER UNIT OR PLACE ON SOFT OR CARPETED SURFACES.

New 2 Year Warranty

On All Silent Alert Products produced after 8th June 2009

9 100% Recycled Paper