




Part of the SA3000 System

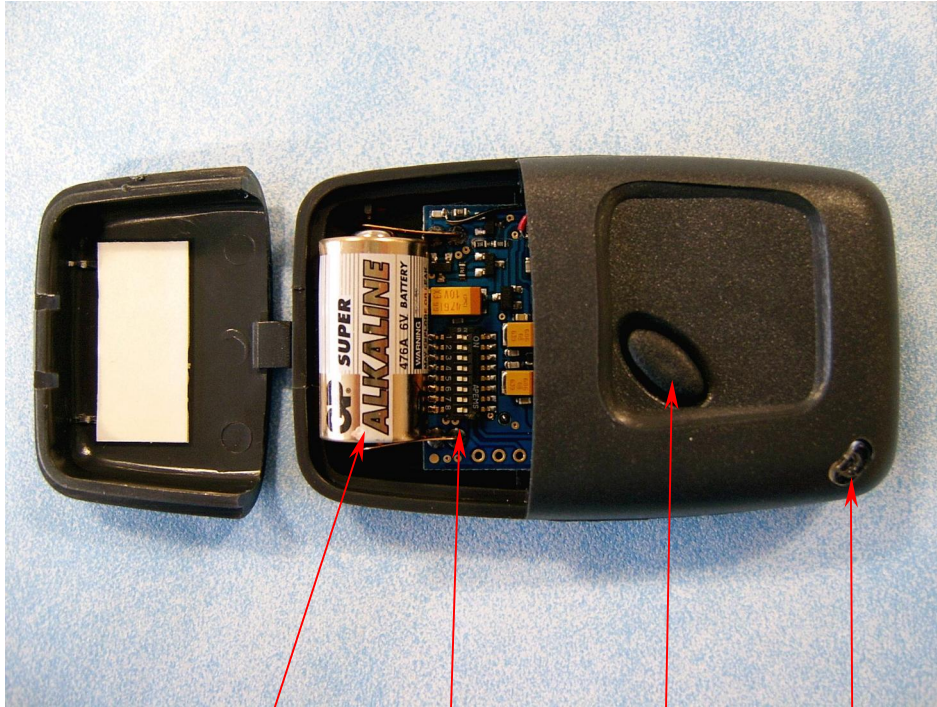

Battery in Battery
Compartment

Code Switches

Send Button

Light

Getting Started

1. Ensure the pager unit is fully charged and either alarm clock charger or trickle charger are installed.
2. Remove battery cover using a small screwdriver & install a 6 Volt Silver Oxide (GP476, 4SR44, 4G13, PX28) battery.
3. Press the send button, the green light should shine on the Key Fob and the person to person key on the Pager or SignWave will illuminate.
4. This device also incorporates a fall alarm. When the device is shaken roughly it will automatically send a radio signal to the pager.
5. The first 4 of the 8 miniature keys adjacent to the battery are for system coding (On away from the battery).
6. A silver Oxide battery should provide a large number of activations. The green light should shine for approximately one second. If the battery is low it will shine orange a low battery signal will be sent.
7. There is a lock out period of 12 seconds after the send button has been pressed.

NOT SUITABLE FOR CHILDREN UNDER 36 MONTHS BECAUSE OF SMALL PARTS. DO NOT IMMERSE THE KEY FOB IN WATER OR ANY OTHER LIQUID. DO NOT SUBJECT THE UNIT TO EXCESSIVE MECHANICAL SHOCK.

New 2 Year Warranty

On All Silent Alert Products produced after 8th June 2009

9 100% Recycled Paper

The SA3000 System comprises a Pager Unit, either an Alarm Clock Charger or Trickle Charger, or a SignWave, a Pillow Pad and at least one Monitor, Key Fob or Movement Monitor. Clofield Ltd.

Help Line 0800-387-397 Txt & Fax 01246-450789 e-mail deaf.info@silent-alert.co.uk