


# KF3A-2219-EU Emergency Person-Person Key Fob

Part of the SA3000 Paging System for deaf and deaf-blind people

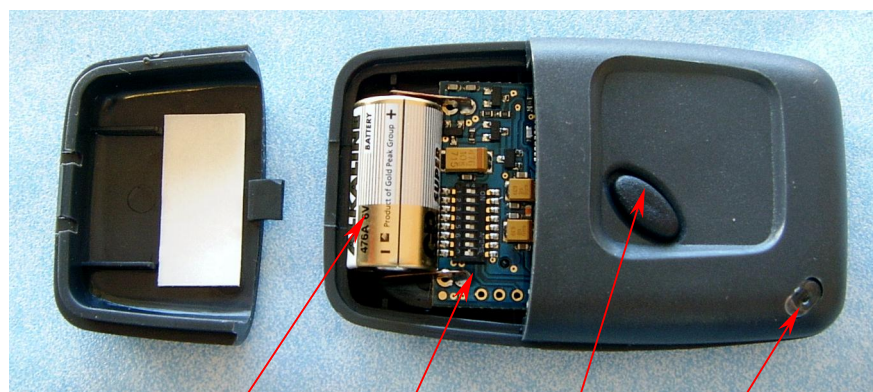

Battery in Battery Compartment    Code Switches    Send Button    Light

FIG 1

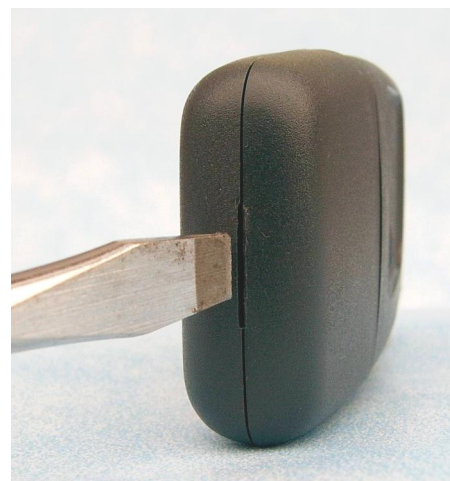


FIG 2

**New 2 Year Warranty**

On All Silent Alert Products produced after 8<sup>th</sup> June 2009

9 100% Recycled Paper

## Getting Started

1. Ensure the pager unit is fully charged and either alarm clock charger or trickle charger are installed.
2. A battery is already installed in the key fob. When the battery needs replacing, remove the battery cover by inserting a small flat blade screwdriver in the slot at the bottom of the key fob and gently twist the screwdriver to release the clip mechanism (see fig 2). Install a 6 Volt Silver Oxide (GP476, 4SR44, 4G13, PX28) battery. A double-sided sticky pad is located in the battery cover and is designed to help locate the battery correctly in the battery compartment (see fig 1)
3. Press the send button, the green light should shine on the Key Fob and the Pager should vibrate and illuminate the Person to Person symbol.
4. The first 4 of the 8 miniature keys adjacent to the battery are for system coding (On, away from the battery).
5. A silver Oxide battery should provide a large number of activations. The green light should shine for approximately one second. If the battery is low it will shine orange and a low battery signal will be sent.
6. There is a lock out period of 12 seconds after the send button has been pressed which will prevent another signal being sent.

05/03/2010

**NOT SUITABLE FOR CHILDREN UNDER 36 MONTHS BECAUSE OF SMALL PARTS. DO NOT IMMERSE THE KEY FOB IN WATER OR ANY OTHER LIQUID. DO NOT SUBJECT THE UNIT TO EXCESSIVE MECHANICAL SHOCK.**

The SA3000 System comprises a Pager Unit, either an Alarm Clock Charger or Trickle Charger, or SignWave, a Pillow Pad and at least one Universal Monitor, Key Fob or Monitor with Smoke Alarm. Clofield Ltd. Help Line 0800-387-397 Txt & Fax 01246-450789 e-mail [deaf.info@silent-alert.co.uk](mailto:deaf.info@silent-alert.co.uk)